EN

NASO LIBERO®

Nasal spray to be used from 6 months

RECOMENDED USE

For daily nasal hygiene.

A non-medicated nasal irrigation spray to help relieve nasal congestion associated with sinusitis, blocked nose, hay fever, allergies and post-operative nasal care.

NASO LIBERO nasal spray is a non-medicated, preservative free, deeply penetrating nasal irrigation spray and can be used on babies, children and adults. It works by thinning mucus and washing away foreign particles such as dust, pollen and allergens. This helps relieve congestion and restore normal respiratory function. NASO LIBERO nasal spray is non habit forming and can be used together with other medications. Frequent sufferers of rhinitis and sinusitis may find that regular use of NASO LIBERO nasal spray may help decrease the use of medicated products.

DIRECTIONS

Hold the can at any angle and gently inset the nozzle into each nostril, press 1-2 seconds.

COMPOSITION

Natural seawater (Diluted to physiological level (0.9%) with Purified Water)

WARNING

- The use of this device by more than one person may spread infections, clean nozzle in soapy water and rinse with hot water after each use
- Pressurized container: May burst if heated
- Keep away from heat / sparks / open flames / hot surfaces. — No smoking
- Pressurized container: Do not pierce or burn, even after use
- Protect from sunlight
- Do not expose to temperatures exceeding 50°C (122°F)
- Keep out of reach of children

125 ml

Look for other **NASO LIBERO®** products

NASO LIBERO monodoses

Distributor:

Linea Trading Ltd.
Poligona bl.9
Sofia 1750, Bulgaria
Tel: +359 (0)2 4439 036
E-mail: office@linea.bg
www.linea.bg

Manufactured by:

Aurena Laboratories AB
Fjärrviksvägen 22
SE-653 50 Karlstad, Sweden

CE 0413

NASO LIBERO Hypertonic Nasal Spray with Aloe and Chamomile

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